
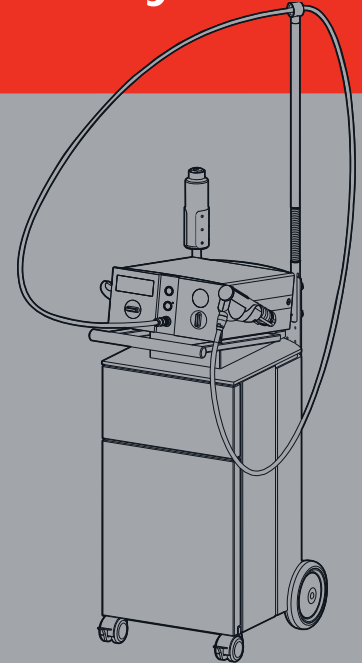
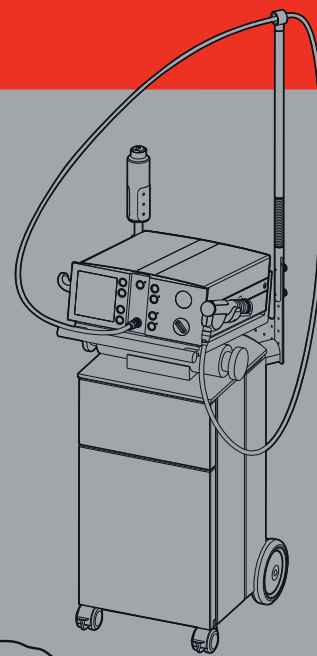
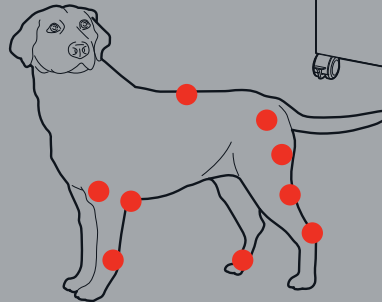
MEDICAL APPLICATIONS  
APPLICATIONS MÉDICALES  
MEDIZINISCHE ANWENDUNGEN  
APLICACIONES MÉDICAS  
APPLICAZIONI MEDICHE  
MEDICINSKA APPLIKATIONER

# Swiss DolorClast® Vet systems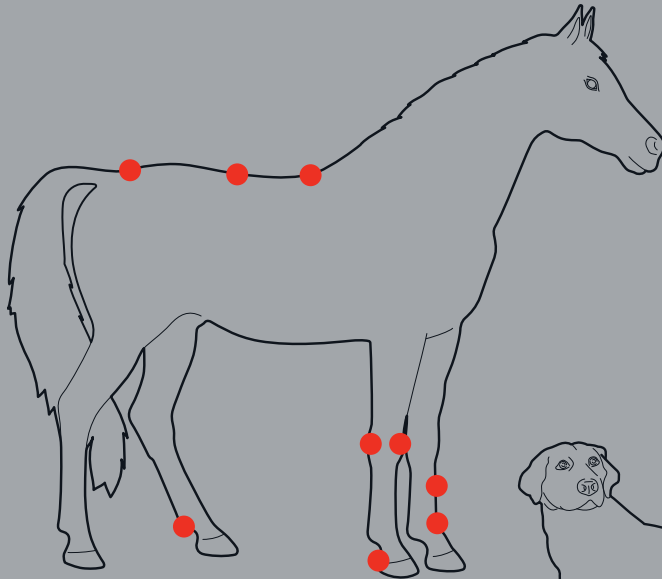

**EMS**   
ELECTRO MEDICAL SYSTEMS

## DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

The following indications have been successfully treated with the EMS Swiss DolorClas<sup>®</sup> Vet in veterinary medicine since 1998:

### Large animals:

- Insertion Desmopathy of the Proximal Suspensory Ligament (high suspensory)
- Insertion Desmopathy of the Distal Suspensory Ligament (low suspensory)
- Osteoporosis (Soft-Bone-Disease) e.g. in the carpal joint
- Calcifications e.g. of the distal suspensory ligaments
- Fractures e.g. of the sesamoid bones
- Dorsal Metacarpal Disease (DMD) e.g. bucked shins
- Exostosis e.g. at cannon bone
- Degenerative Joint Disease (DJD) e.g. ring-bones
- Navicular Disease
- Hindlimb Desmitis e.g. patella ligament
- Dorsal Spinal Processes (DSP) e.g. Kissing Spine
- Back Pain
- Caudal Heel Pain
- Sacroiliac Pain
- Angular limb Deformities

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# PLEASE READ THIS BEFORE BEGINNING WORK!

## About this manual

Note that the English version in this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

This operating instructions describes the EMS Swiss DolorClas<sup>®</sup> Vet treatment of chronic musculo-skeletal disorders and myofascial pain syndromes in small and large animals.

Please read these operating instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

**Danger**  
Risk of injury

**Caution**  
Risk of property damage or environmental harm

**Please note**  
Useful additional information and hints

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### Förbjudet

### Tillåtet

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problem. Please call directly in your approved EMS repair center or with your dealer.

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EMS

## **Technical modifications**

EMS reserves the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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## CONTENTS

Treatment.....	2
General information .....	2
Contraindications .....	4
Side effects .....	6
General hints .....	8
Therapy .....	10
Working technique for horse.....	10
Operating parameter for the horse .....	14
Horses indications .....	16
Working technique of the dog .....	26
Operating parameter for the dog .....	30
Dog indications .....	32
TST- Trigger-Shockwave-Therapy®	
Myofascial pain syndrome .....	36
Working technique .....	38
Safety precautions.....	40

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## General information

Extracorporeal Shockwave Therapy (ESWT) / Radial Shockwave Therapy (RSWT)

Shockwaves have revolutionised the worlds of urology and orthopaedics. They allow non-invasive procedures.

Following the introduction of shockwaves in the treatment of urinary calculi in the ureters in the early 1980's, they soon gained worldwide recognition. The use of shockwaves quickly became an efficient method of treatment in urology. Today, some 70 % of all patients who have to undergo treatment for kidney or ureteral stones are treated with extracorporeal shockwaves (ESWL = Extracorporeal Shockwave Lithotripsy).

At the same time, research was underway into other indications in oncology, surgery, gastroenterology and orthopaedics. Because of highly promising results, the studies focussed increasingly on orthopaedics. During a research project on the side effects of shockwaves, a Bulgarian/German research team discovered the healing effect of shockwaves in the treatment of bone fractures. It was also noticed that, following extracorporeal shockwave therapy, many patients became pain-free in the treated area. This finding prompted its use in the treatment of pain, particularly for the treatment of chronic musculoskeletal disorders.

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## Contraindications

Treatment with this product may not be implemented in the following cases:

- Thrombosis
- Blood-clotting disorders (haemophiliacs)
- Ingestion of medicinal products that affect blood clotting
- Acute inflammation in the treatment area
- Tumours in the treatment area
- Gestation
- Gas-filled tissue (lung tissue) in the immediate treatment area
- Major vessels and nerve tracts in the treatment area

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## Side effects

The following side effects of the treatment with the EMS Swiss DolorClast® Vet have been observed: irritation, petechiae, hematoma, swelling and pain. The effects disappear relatively quickly (i.e. 1 - 2 weeks). Skin lesions could be observed if patients were pre-treated with cortison for a longer time period.

With TST - Trigger-Shockwave-Therapy®, petechiae and mild bruising are possible, depending on the amount of application force. The altered biomechanics may cause symptoms to emerge temporarily in other parts of the body.

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## General hints

 Please make sure that all side effects have subsided before repeating the treatment. An analgesic effect sets in after approx. 500 pulses. As treatment continues, the pressure output (bar) and operating frequency (Hz) can then be increased to the recommended operating parameters. If the sensitivity of the patient to pain does not allow this, reduce the application force of the handpiece slightly or use a local anaesthetic.

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## THERAPY

(EXCEPTED TST - TRIGGER-SHOCKWAVE-THERAPY®)

### Working technique for horse

Sedation of the equine patient is recommended.

For different indications, please find the suitable applicator according to the Quicktable.

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- The site of treatment is localized by palpation,
- After localization, clean the corresponding area,
- Use EMS Swiss DolorClast® coupling gel for improved shockwave transmission,
- Gently rub in multiple pulse mode with the applicator tip over the site of treatment. The exerted application force varies and is depending on the indication.

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## Operating parameter for the horse

Recommended operating parameters	Pathologies of the musculoskeletal system.	Myofascial pain syndromes
Pulse count	2000*	2000 - 6000
Operating pressure	2 – 4 bar	2 – 4 bar
Operating frequency	6 - 10 Hz	6 - 15 Hz
Number of sessions	3 - 5	6 - 10
Treatment interval	7 - 10 days	3 - 10 days
Recovery phase post final treatment	2 - 6 weeks	

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## Operating parameter for the dog

Recommended operating parameters	Pathologies of the musculoskeletal system	Myofascial pain syndromes
Pulse count	1000 - 2000	2000 - 3000
Operating pressure	1.5 - 2 bar	2 bar
Operating frequency	6 - 10 Hz	6 - 15 Hz
Number of sessions	2 - 5	4 - 10
Treatment interval	7 - 10 days	3 - 10 days
Recovery phase post final treatment	2 - 4 weeks	

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Tendiniter t ex bicepstendinit

## TST- TRIGGER-SHOCKWAVE-THERAPY® MYOFASCIAL PAIN SYNDROME

Use of the EMS Swiss DolorClas<sup>®</sup> Vet in treatment of myofascial pain syndromes requires specialist training with proof of certification. For further details of training events, please contact the manufacturer.

Without in-depth training in diagnosis and treatment, the use of the EMS Swiss DolorClas<sup>®</sup> Vet may be dangerous to the patient and will afford no guarantee of the treatment results being successful, as desired.

A correct diagnosis must be established before applying the shockwaves.

The symptoms and triggerpoint locations are not in the same part of the body. Symptom-orientated treatment is not successful with triggerpoint related pain syndromes.

The recommended operating parameters for the various triggerpoints in the body and the precautions for preventing complications are explained at the certification seminars.

The pulse count to be applied to each trigger point depends on the size and depth of a triggerpoint.

The operating parameters vary according to the area of use and the muscle region. Please note that no more than 4000 – 6000 pulses should be emitted in per session.

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## **Working technique**

- The treatment area is determined before hand by means of a triggerpoint diagnosis. An ultrasound diagnosis can also be carried out to differentiate between the muscle layers and to establish the precise location within the muscle. Please use biofeedback of the patient to identify the affected muscle
- Use EMS coupling gel for improved coupling
- Stroke the applicator over the site of treatment

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## SAFETY PRECAUTIONS

EMS and the distributor of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operating instructions, or improper preparation and maintenance.

 For further details on the preparing, operating and maintenance of this product, please refer to the relevant operating instructions of the unit and handpiece kit.

 Use for the intended purpose only: Before using the product, make sure that you have studied and understood the operating instructions. This also applies to any equipment used with this product. Failure to observe the operating instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel.

 Before each use, always clean, disinfect, rinse and sterilize the different pieces and accessories of this product. Please refer to the information provided in these operating instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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EMS Corporation
EMS Medical GmbH